HEALTHCARE SOLUTION™

Ultrasonic Nebulizer
초음파 흡입기
NB-152U

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Thank you for your purchase.


For the proper using & maintaining the scale please read this instruction completely before operating.

TABLE OF CONTENT


PARTS AND COMPONENTS	15
SYMBOLS	16
GENERAL	17
IMPORTANT SAFEGUARDS	
- PREPARATION	17
- OPERATION	18
- CLEANING	18
- STORAGE	18
OPERATION PROCEDURE	
- FILL WATER RESERVOIR	19
- FILL MEDICATION CUP	19
- CONNECT EXTERNAL POWER	20
- TURN ON THE POWER AND ADJUST NEBULIZING RATE	20
- ADJUST THE AIR FLOW RATE	21
- ASSEMBLE THE MASK (OR MOUTHPIECE) AND BEGIN TREATMENT	21
- FINISH OPERATION	21
DISPOSE EXCESS MEDICATION AND WATER	22
CLEANING	22
SPECIFICATIONS	23
TROUBLESHOOTING	23
WARRANTY	24

PARTS AND COMPONENTS

Main Body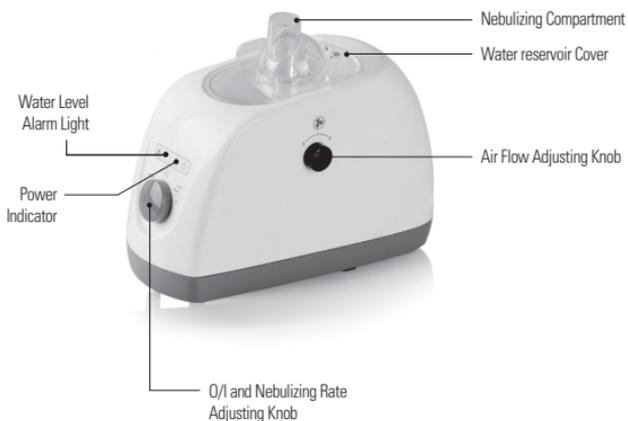

Nebulizing Compartment

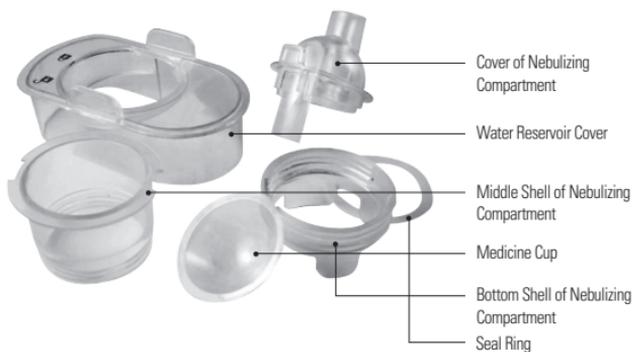SYMBOLS

Symbols	Meaning
	Manufacturer
	Authorized Representative in the European community
	Symbol for the marking of electrical and electronics devices according to Directive 2002/96/EC. The device, accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal
	CE marking in conformity with EC directive 93/42/EEC
	Keep dry
	Attention, consult accompanying documents
	Type B Applied Part

GENERAL

This desk type ultrasonic nebulizer (Hereinafter: equipment) is a medical device that delivers liquid medication in the form of fine, dense aerosol.

The nebulizer gas flow transports the wet aerosol to the body through windpipe and lung so as to be better absorbed by the patient.

IMPORTANT SAFEGUARDS

Read all Instructions carefully before use!

PREPARATION

- Use the equipment only for its intended use as described in this manual.
- Use the equipment with medications only under the instruction of your physician.
- Do not use the equipment if it has any damaged parts, or it has fallen into water. Return the product to a service center for examination and repair.
- Always unplug this product before filling and after use.
To use the equipment for the first time, or after storing it for an extended period, be sure to clean and sterilize all necessary parts as described on page 22.
- Do not handle the power cord with wet hands or immerse the AC adapter. An electrical shock may result.
The medication cup and mask (or mouthpiece) should be checked before each use. Replace with a new one if they are dirty or worn out.
- Do not fill medication cup over 12ml or the nebulizing rate will be weak.

OPERATION

- Close supervision is required when this equipment is used by children or invalids.
- Stop using the equipment immediately if any abnormality occurs.
- Keep your eyes away from the output of medication mist.
- Do not use the equipment where flammable gas is being used.
- Never operate the equipment with empty water tank or empty medication cup.
- Do not use solutions other than water for the tank. It may damage the transducer.
- Do not tilt or shake when operation.
- Never open the water reservoir cover or nebulizing compartment cover during operation. It may cause a severe injury.
- Some medications may not have good atomization.
Check with the physician or manufacturer of medication.

CLEANING

- Never immerse the equipment in water. It may damage the unit.
- Clean all necessary parts after each use as described on page 22

STORAGE

- Do not store the unit under direct sunlight, high temperature or humidity.
- Keep the unit out of reach of small children.
- Always keep the unit unplugged while not in use.

OPERATION PROCEDURE

- To use the equipment for the first time, or after storing it for an extended period, be sure to clean and sterilize all necessary parts as described on page 22.
- Always unplug this product and turn off the switch at bottom before filling and after use.

FILL WATER RESERVOIR

- Take out the cover of water reservoir as Fig. 1 (Nebulizing compartment will be taken out as a whole as well), and don't add water over the water level indicator as Fig. 2.
- Fill water reservoir with tap water or distilled water.

Fig. 1

Don't add water over water level indicator

Fig. 2

Caution

- Use of cool water will result low nebulizing rate.
- For higher nebulizing rate, use warm water (over 30°C).
- Do not use solutions other than water for the reservoir. It may damage the transducer.

FILL MEDICATION CUP

- Screw off the cover of nebulizing compartment as Fig. 3 (If you want to clean or change the medicine cup, please screw off the bottom shell of nebulizing compartment as Fig. 5 and Fig. 6).
- Fill medication directly into medication cup as Fig. 4.
- Replace the cover of nebulizing compartment.

Fig. 3

Fig. 4

Fig. 5

Fig. 6

⚠ Caution

- Do not fill more than 12ml, otherwise the nebulizing rate will be affected.
- Use medications only under instructions of your physician.

CONNECT EXTERNAL POWER

Plug the power cord into an AC outlet.

TURN ON THE POWER AND ADJUST NEBULIZING RATE

- A. Make sure water reservoir and medication cup are properly filled, and the unit is placed on a flat and dry surface.
- B. Turn the O/I Knob from 'off' position to the range between MIN MAX, and the power indicator should be lit. Screw to the reasonable position to get your favorite nebulizing rate.

ADJUST THE AIR FLOW RATE

Turn the 'Air Flow Adjusting Knob' in the range between MIN and MAX to get your favorite air flow rate.

ASSEMBLE THE MASK (OR MOUTHPIECE) AND BEGIN TREATMENT

Assemble the extension tube and mask (or mouthpiece) to the nebulizing compartment as Fig. 7 and Fig. 8 to carry out a treatment.

Fig. 7

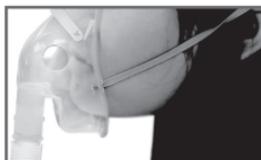
Fig. 8

Caution

- Stop operation immediately if any abnormality is noted .
- When little water is left in the water reservoir, the equipment will turn off automatically and water level indicator will be lit to remind the user to add water.

FINISH OPERATION

- A. Turn off the equipment by switching the O/I button to the 'off' position.
- B. Unplug the power cord from AC outlet.

DISPOSE EXCESS MEDICATION AND WATER

- A. Take out the cover of water reservoir together with the nebulizing compartment.
- B. Remove the cover of the nebulizing compartment.
- C. Pour remaining medication out of medicine cup.
- D. Drain water from the water reservoir.

Caution

- Do not touch water, it is hot after use.

CLEANING

- A. Mask, mouthpiece, extension tube, water reservoir cover, nebulizing compartment, medication cup: Clean with mild detergent or commercially available disinfectant. Rinse all parts thoroughly and let air dry.
- B. Main body: Clean with a soft cloth moistened with water and non-abrasive cleanser.

Caution

- Be sure to clean and sterilize all necessary parts after each use.
- Always unplug this product before cleaning.
- Never immerse the equipment in water. It may damage the unit.

SPECIFICATIONS

Product Name	Ultrasonic Nebulizer (NB-152U)
Power	220Vac 60Hz, 230Vac 50Hz
Power consumption	Approx. 45VA
Safety classification	Type B apparatus. Class II
Ultrasonic frequency	1.7MHz \pm 10%
Max. nebulizing rate	\leq 1.5ml/min
Particle size	\leq 5 μ m
Medication cup capacity	12 ml
Noise Level	\leq 55DB
Operation temperature and humidity	+10°C to +40°C, 75% and below
Storage temperature and humidity	-10°C to +40°C, 95% and below
Dimensions	245mm x 100mm x 190mm
Weight	Approx. 1400g (without accessories)
Accessories	Extension tube (2 pieces), Medicine cup (5 pieces), Child & Adult masks, Mouthpiece (1 pieces), sealing ring (1 pieces).

TROUBLESHOOTING

Symptom	Cause and/or solution
Unit does not operate when O/I button is turned on	Check that power cord is plugged in firmly.
	Check that enough water is filled to the water reservoir.
No atomization or low atomization rate	Check that there's medication in the medicine cup.
	Check that the medication cup is not broken or transfigured.
	Check medication level in medication cup.
	Check that transducer is clean.

The manufacturer reserves the right to make technical changes without notice in the interest of progress. Prior notices will not be given in case of any amendments within this manual. The mentioned trademarks and names are owned by the corresponding companies .

WARRANTY

- CAS warrants its products free of defects in materials and workmanship in normal use for a period of ONE years from the date of retail purchase.
- This warranty does NOT cover damages caused by misuse or abuse, including but not limited to:
 - Failure caused by unauthorized repairs or modifications;
 - Damage caused by shock or drop during transportation;
 - Failure caused by improper operation inconsistent with the instructions stated in this user manual;
 - Malfunction or damage from failure to provide the recommended maintenance;
 - Damage caused by improper use of power supply.

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